DSD 1st APPOINTMENT FACIAL SCANNER PROTOCOL

he new DSD 1st appointment protocol

The new DSD 1st appointment protocol

Agregar aqui video de angel y protocolo de fotos

Scans required

1. Facial scan in maximum smile:

- The patient should smile exaggeratedly and constantly.
 - It is recommended that the patient bite an ejector in the posterior sectors for stability.
- Both ears must be visible

Scans required

2. Facial scan at rest:

- The patient should keep the mouth slightly open. (Lips not touching)
- Both ears must be visible

Photo/Video DSD Protocol


5 Photos

1 Video

From the 2 facial scans we can get the DSD photo protocol and much more

Files overlapping and design process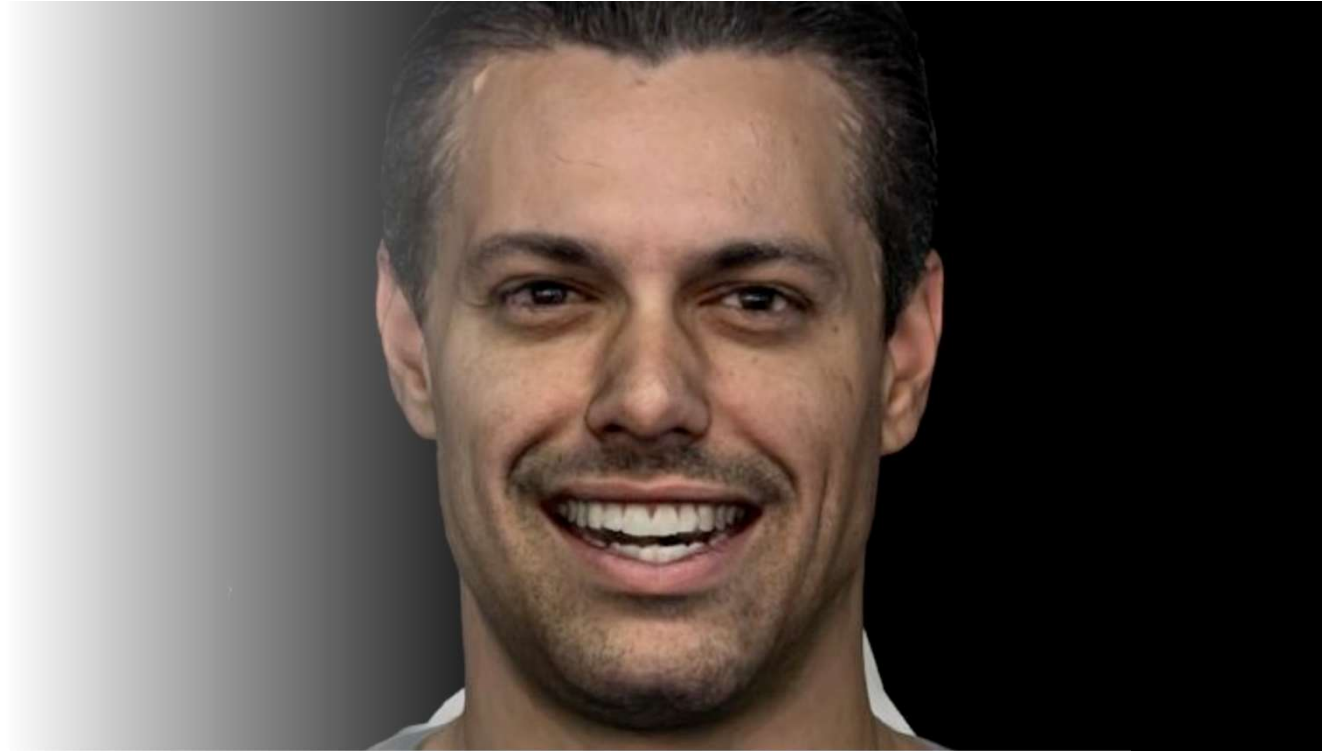

3. Scans required

1. Facial scan in maximum smile:

- The patient should smile exaggeratedly and constantly.
- It is recommended that the patient bite an ejector in the posterior sectors for stability.

3. Scans required

2. Facial scan at rest:

- The patient should keep the mouth slightly open. (Lips not touching)


1. Materials

- Iphone 12 and above
- Qlone Dental Pro app
- Adequate light source
- Ejector-type teether
- Uniform background
- Good internet connection


The ideal lighting

- The best lighting for scanning is **natural light**, as it provides uniform illumination. Ideally, position the patient near a window, with the light source behind the operator and facing the patient.
- If using **artificial light**, it should be evenly distributed and positioned in front of the patient, avoiding shadows that could compromise scan quality.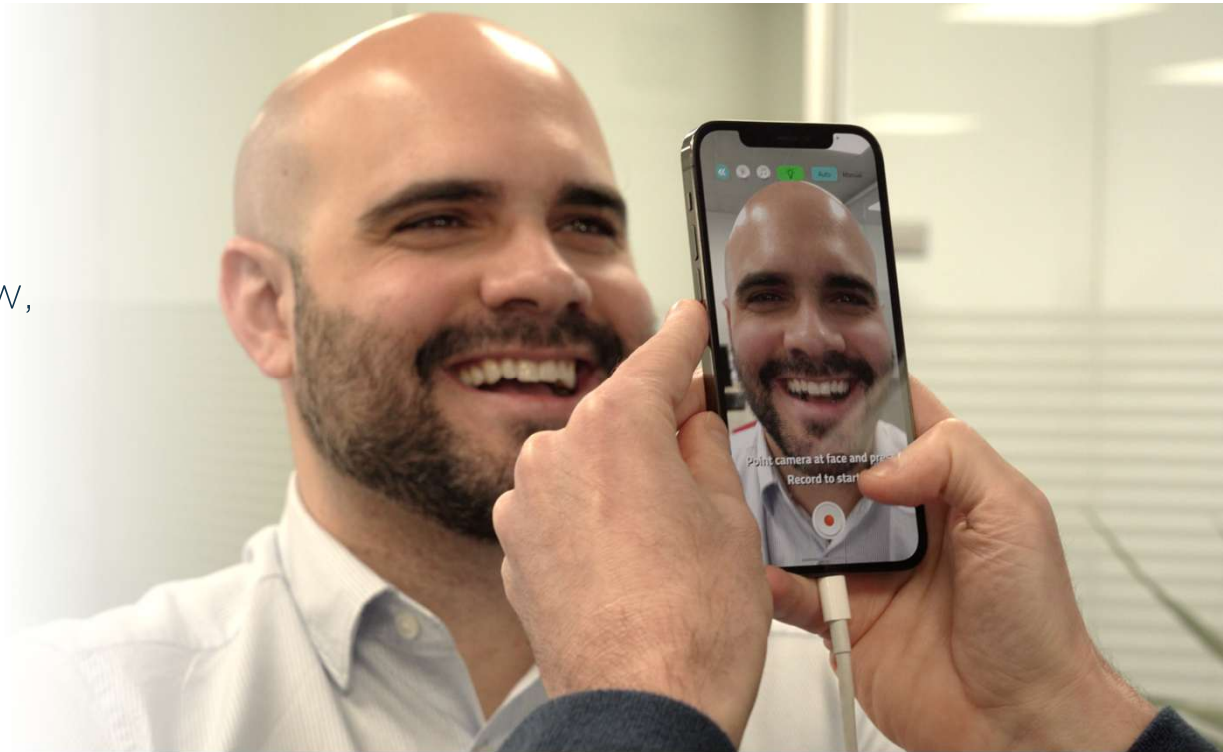

The ideal background

- Additionally, the background should be uniform and free of distractions, as the scanner may interpret elements in the background as part of the face. A plain white background is ideal, ensuring the patient is well-separated from it.

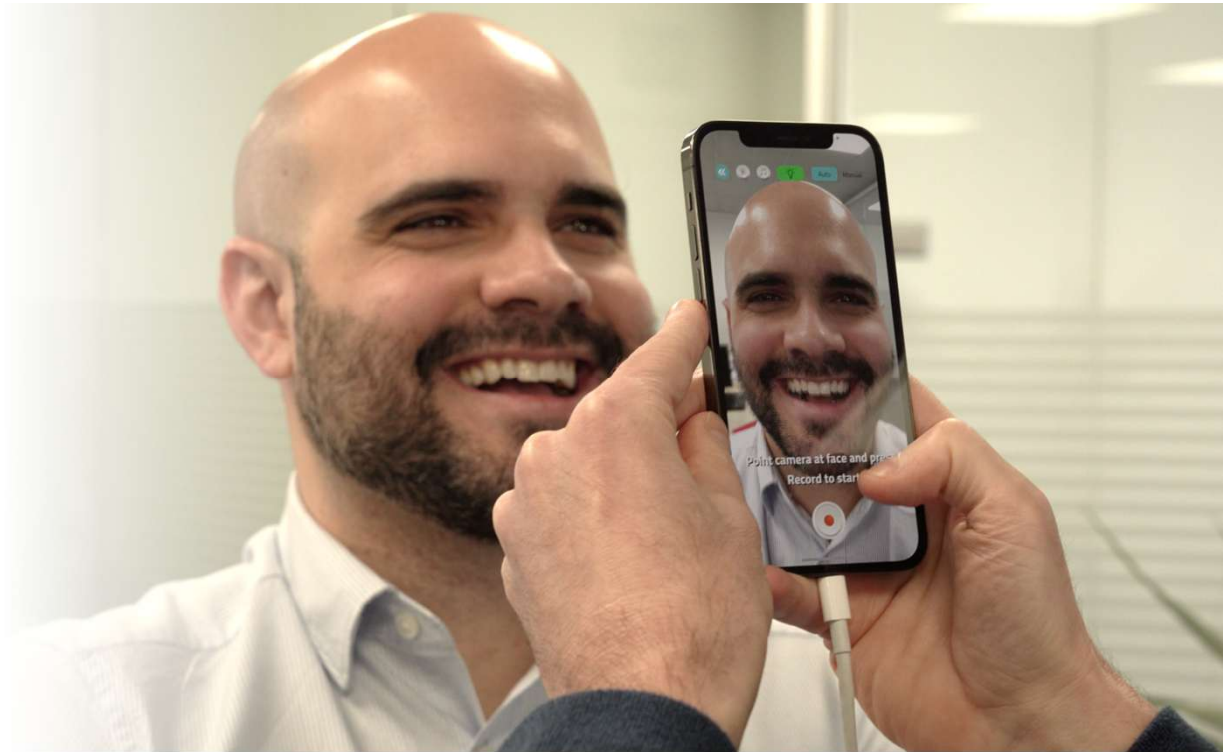2. Facial Scan Capture

1. Open the **Qlone Dental app** and make sure the light bulb on the screen is **green**.
 - If it is not green, adjust the position of the phone or change the lighting.
 - If you tap the light bulb, the light level will be displayed (it should be above 150).

2. Facial Scan Capture

2. Patient and operator position

- The operator stands in front of the patient, approximately 30 cm away.
- The patient's face should occupy the entire screen of the device.

2. Facial Scan Capture

3. Scanning Sequence

1. Main Movement

Start the capture with the record button.

2. Lateral movement:

- From the center to the left until you are in a perpendicular position.
- Return to the center.
- Go to the right until you are in a perpendicular position.

Always keeping the same distance

2. Facial Scan Capture

3. Scanning Sequence

1. Main Movement

Start the capture with the record button.

2. Lateral movement:

- From the center to the left until you are in a perpendicular position.
- Return to the center.
- Go to the right until you are in a perpendicular position.

Always keeping the same distance

2. Facial Scan Capture

3. Vertical movement (half moon):

- From the right perpendicular position, start the half moon movement from right to left, with the scanner at a lower angle, capturing the jaw, jowls and neck.
- Complete the reverse half moon from left to right, now from a higher angle, capturing the forehead and top of the head.

2. Facial Scan Capture

4. Capture dental details (if frames are available):


- Bring the scanner closer to the mouth (10 cm distance).
- Capture images manually in the following sequence:
 - Left perpendicular → diagonal → center → diagonal → right perpendicular.

2. Facial Scan Capture

5. Tips for Optimal Scanning

- ✓ Maintain fluid and constant movement.
- ✓ Always keep the same distance between the device and the patient.
- ✓ Avoid sudden changes in light during the capture.
- ✓ Ask the patient not to move their head while the scan is being performed keeping their gaze on a fixed point
- ✓ Make sure both ears are visible

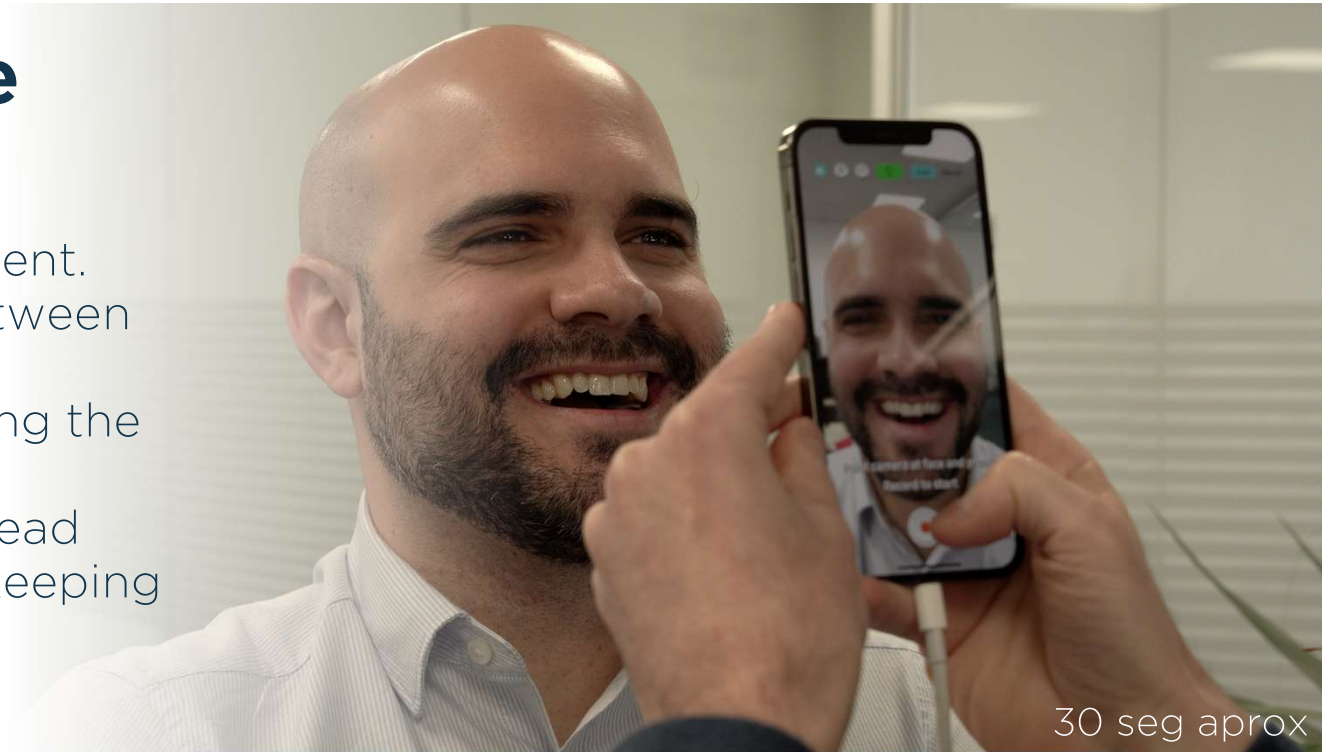The Result

The Result


3. Export scan

1. Process the image in the application.
2. Export in PLY for compatibility with the DSD Planning Center.


1. Using your iPhone as a server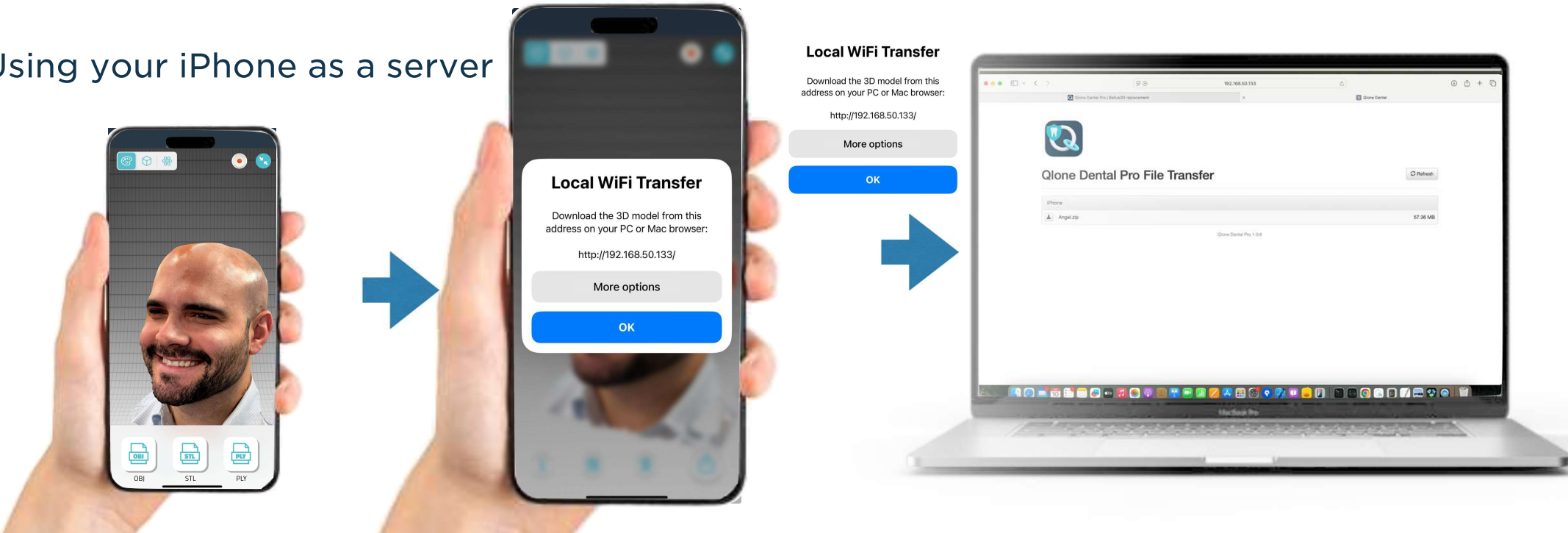

1. Traditional methods

- Airdrop
- Email
- Message

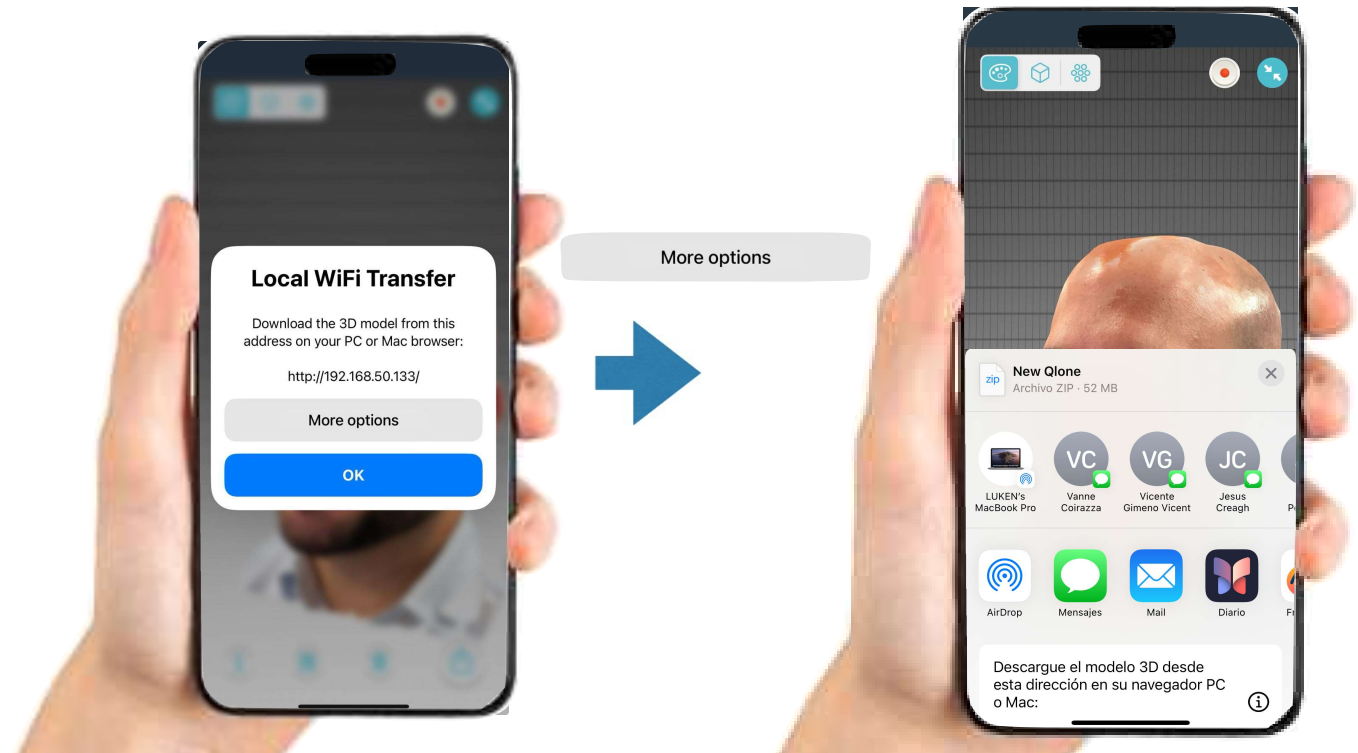Files overlapping and design process

Reviewing your cases

DSD 1st APPOINTMENT FACIAL SCANNER PROTOCOL

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